Technische Daten des Gehörschutzes epro-X.S nach EN 352-2:2020
Technical data of the hearing protection epro-X.S in accordance with EN 352-2:2020
Données techniques de la protection auditive epro-X.S selon la norme EN 352-2:2020
Dati tecnici dell'otoprotettore epro-X.S conforme a EN 352-2:2020
Datos técnicos de la protección auditiva epro-X.S conforme a EN 352-2:2020

Table with columns: Hz, 63, 125, 250, 500, 1000, 2000, 4000, 8000. Rows: epro-X.S, epro-X.S BTE X.5M, epro-X.S, epro-X.S BTE X.4M.

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User instructions for hearing protection epro-X.S
With your new, custom-made hearing protection epro-X.S you have decided to protect your hearing against damaging or disturbing noise by using custom-made protectors.

Prior to using your hearing protection device
Please read these instructions carefully to familiarize yourself with the proper handling of the hearing protection device.

Table with columns: Hz, 63, 125, 250, 500, 1000, 2000, 4000, 8000. Rows: epro-X.S, epro-X.S BTE X.5M, epro-X.S, epro-X.S BTE X.4M.

Table with columns: Hz, 63, 125, 250, 500, 1000, 2000, 4000, 8000. Rows: epro-X.S, epro-X.S BTE X.3M, epro-X.S, epro-X.S BTE X.2M, epro-X.S, epro-X.S BTE X.1M*.

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Additional features of your hearing protection device
• Connecting cord (detachable connection between the right and left hearing protector)
• Safety clip to attach connecting cord to clothes

Optional additional features and products
• Attenuation elements with different attenuation effect
• egger care products for cleaning and disinfecting

Your hearing protection device is equipped with a sound canal, which contains an attenuation element. Depending on your requirements, you may use other attenuation elements.

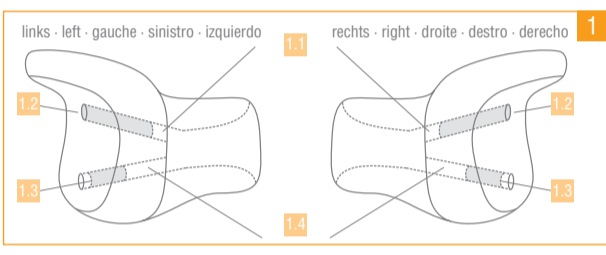
- Inserting and removing your hearing protection device (see figure 2 to 4)
1. With clean hands, take the hearing protection epro-X.S out of the packaging.
2. Make sure that your entire hearing protectors have not been damaged and are clean.

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

- 5. Take the hearing protector to your ear without twisting it. Angling it from behind, insert it into the auditory canal (figure 3). Be sure to insert the hearing protection devices deeply enough and position it comfortably (figure 4).
6. To facilitate the insertion of the protectors, we recommend using cedis care gel (see additional products).

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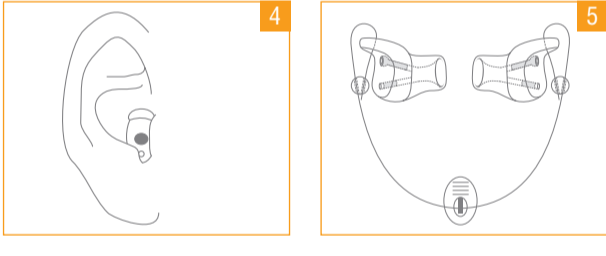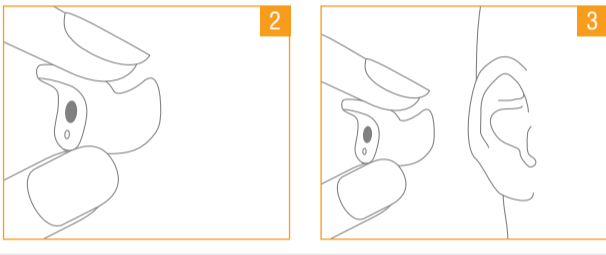- WARNING
A) Noise impact on hearing without hearing protection
Even a brief exposure to a high noise level while wearing improperly inserted hearing protectors or no hearing protection devices at all may damage your hearing.
B) Protective effect that is too low when the hearing protector is incorrectly inserted in the ear

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- E) Reduced perception
Please consider: While wearing the hearing protection devices, the signals in the surrounding area will also be muffled (e.g. warning signals).
F) Risk of injury with improper handling
Do not pull the hearing protectors out of the ear by the connecting cord. Sudden or fast removal of the ear plugs out of the auditory canal may damage the ear drum.

- 1.1 Zusatzkanal
1.2 Verschlussstopfen
1.3 Dämmelement
1.4 Schallkanal
1.1 additional canal
1.2 sealing plug
1.3 attenuation element
1.4 sound canal

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Storage and care
This product may suffer if certain chemical substances are used. Your hearing protectors must be cleaned regularly to avoid a decrease in its protective effectiveness.

ATTENTION
These products may not be altered or modified. If the suggestions in these instructions are not adhered to, the effectiveness of the hearing protection device may be affected considerably.

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CE Declaration of Conformity
epro-X.S hearing protection devices are manufactured individually after ear impression taking. Tested and monitored by PZT GmbH, Bismarckstrasse 264 B, D-26389 Wilhelmshaven.